



# Steam Sensor S

According to  
**RKI/KRINKO 2012**  
Appendix 4  
Our recommendation: Place the test in the chamber  
to monitor each sterilization cycle.

Conform to  
**DIN EN ISO**  
11140-1 Typ 5

**Special chemical indicator system for monitoring steam sterilization  
in autoclaves with N and S programs**

## Product details


The Steam Sensor S from Stericop is a Type 5 indicator system for monitoring N and S sterilization cycles. It is ideal for verifying „wet“ sterilization cycles, such as those in Statim cassette autoclaves, Euroklav units, and rapid programs.

## Application

1. Place the Steam Sensor S in the sterilization chamber and start the cycle.
2. After the cycle ends, remove the indicator and evaluate visually.

## Evaluation

Check the color change against the reference chart inside the packaging. A clear shift from blue to pink indicates successful sterilization of the medical devices. Record the result in the sterilization logbook.

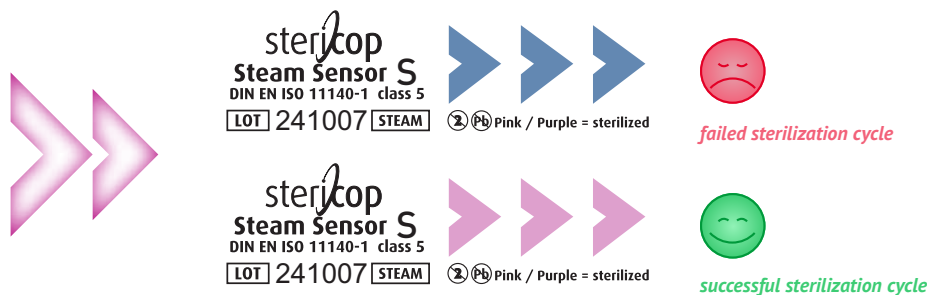

Stericop GmbH & Co. KG  
Biedrichstraße 10  
61200 Wölfersheim

T +49 (0) 6036 984 33 0  
F +49 (0) 6036 984 33 21  
info@stericop.de  
www.stericop.de

Product	Article number	Package size	Storage	Shelf life
Steam Sensor S	212002	200 pieces	dark & dry at room temperature	3 years from date of manufacture